



Szeroka matryca liniowa 13L4w


9011







| | Środek czyszczący | Zgodność | Typ zgodności | Komentarze |
|--------------------------|--|----------|-----------------------------------|------------|
| Czyszczenie ^d | 3E-Zyme (czyszczenie ręczne) | tak | Zatwierdzony | - |
| | 3E-Zyme (użycie automatyczne) | tak | Zgodny z materiałem ^{za} | - |
| | Bodedex forte | tak | Zgodny z materiałem ^{za} | - |
| | Wieloenzymatyczny środek myjący CIDEZYME XTRA / Enzymatyczny środek myjący CIDEZYME GL | tak | Zgodny z materiałem ^{za} | - |
| | Gigazyme | tak | Zgodny z materiałem ^{za} | - |
| | Intercept Wipes / Intercept Detergent | tak | Zgodny z materiałem ^{za} | - |
| | Korsolex Endo-Cleaner 0,5% | tak | Zgodny z materiałem ^{za} | - |
| | MATRIX Biofilm Remover | Yes | Material Compatible ^a | - |
| | Neodisher MediClean Forte | Yes | Material Compatible ^a | - |
| | pH neutral (pH 6-8), non-corrosive cleaning products intended for medical devices | Yes | Material Compatible ^a | - |

| | | | | |
|--|--|-----|---|------------------------|
| | Prolystica 2x concentration. Enzymatic | Yes | Material Compatible ^a | – |
| | Revital-Ox Bedside Complete/2X Concentrate Enzymatic Detergent/Enzymatic Detergents | Yes | Material Compatible ^a | – |
| | Sekusept MultiEnzyme P | Yes | Material Compatible ^a | – |
| | Suma Med Enzyme | Yes | Material Compatible ^a | – |
| | Medivators Advantage Plus Intercept (detergent), Rapidice PA Disinfectant flush: 70% isopropyl alcohol | Yes |  Validated | Automated disinfection |
| | Korsolex Basic (Manual Disinfection) | Yes |  Validated | – |
| | Tristel Fuse for Instruments (Tristel Fuse for Stella) | Yes |  Validated | Manual Disinfection |
| | Revital-Ox Resert/Resert XL HLD | Yes |  Validated | Manual Disinfection |
| | Ethanol 70% (wiping) | Yes |  Validated | Manual Disinfection |
| | Accel Prevention (wipes, ready-to-use liquid, concentrate) | Yes | Material Compatible ^a | – |
| | Adaspor Single Shot | Yes | Material Compatible ^a | – |
| | Antigermix S1 | No | – | – |
| | Astra VR (with approved disinfectant) | No | – | – |

Disinfection ^d

| | | | |
|--|-----|----------------------------------|---|
| Bomix Plus | Yes | Material Compatible ^a | – |
| Cavi Wipes/CaviCide | Yes | Material Compatible ^a | – |
| Cidex OPA | Yes | Material Compatible ^a | – |
| Cleanisept Wipes Forte | Yes | Material Compatible ^a | – |
| Gigasept FF | Yes | Material Compatible ^a | – |
| Glutaraldehyde 2% - 3.4% | Yes | Material Compatible ^a | – |
| Incidin OxyFoam/Incidin OxyWipe S | Yes | Material Compatible ^a | – |
| Isopropanol 70% (wiping) | Yes | Material Compatible ^a | – |
| Korsolex Basic (Automated use) | Yes | Material Compatible ^a | – |
| Korsolex Endo Disinfectant 1%/Korsolex Extra | Yes | Material Compatible ^a | – |
| Meliseptol Foam | Yes | Material Compatible ^a | – |
| Meliseptol Wipes Sensitive | Yes | Material Compatible ^a | – |
| Metricide OPA Plus | Yes | Material Compatible ^a | – |
| Mikrobac Tissues | Yes | Material Compatible ^a | – |
| Mikrozoid AF Liquid | Yes | Material Compatible ^a | – |

| | | | |
|--|-----|--|---|
| Neodisher Endo Sept GA | Yes | Material Compatible ^a | – |
| Neodisher Endo SEPT PAC/Neodisher Septo PAC | Yes | Material Compatible ^a | – |
| Neodisher Septo DN | Yes | Material Compatible ^a | – |
| Nu-Cidex | Yes | Material Compatible ^a | – |
| OPAL | Yes | Material Compatible ^a | – |
| Rapicide/Rapicide OPA/28 | Yes | Material Compatible ^a | – |
| Rely+On Perasafe | Yes | Material Compatible ^a | – |
| Revital-Ox Resert / Resert XL HLD (Automated use) | Yes | Material Compatible ^a | – |
| Sani Cloth Plus Wipes/Sani Cloth Super Wipes | Yes | Material Compatible ^a | – |
| Sekusept Aktiv | Yes | Material Compatible ^a | – |
| Steranios 2%, 2% N.G., 2% E.C.S | Yes | Material Compatible ^a | – |
| Thermosept PAA | Yes | Material Compatible ^a | – |
| Tristel Duo for Ultrasound/Tristel Trio Wipe System | Yes | Material Compatible ^a | – |
| Trophon EPR / Trophon 2 | Yes | Material Compatible ^a | Fit transducer completely above line mark. |
| STERIS System 1, 1E, 1 Plus and 1 Express ^{c, d} | Yes |  Validated | – |

| | | | | |
|---------------|---|-----|---|---|
| Sterilization | STERIS V-Pro 1 Plus, V-Pro 60Non lumen cycle ^c | Yes |  Validated | – |
| | STERIS V-Pro maXNon lumen cycle or Flexible cycle ^c | Yes |  Validated | – |
| | Sterrad NX & 100NXStandard cycle ^c | Yes |  Validated | – |
| | Sterrad 100 NX Express Cycle | Yes |  Validated | – |
| | Sterrad 100SOne cycle only (USA),Short cycle (rest of the world) ^c | Yes |  Validated | – |
| | Sterrad 200Short cycle ^c | Yes |  Validated | – |
| | Matachana 130LF, Webeco FA90, Webeco FA95 (60°C cycle) | No | – | – |
| | RENO-20/RENO-30/RENO-D50: ECO Cycle | Yes | Material Compatible ^a | – |
| | RENO-S90, RENO-S130, RENO-S130D: ECO Cycle | No | – | – |
| | RENO-S90, RENO-S130, RENO-S130D: Non Lumen Cycle | Yes | Material Compatible ^a | – |

Footnotes: *Follow local regulations for minimum reprocessing. Check table 4 on page 29 of the Care and Cleaning Guide.

*Cleaning is an important first step for all transducer types. Every transducer used in a critical procedure must be sterilized as the final reprocessing step.

a) This table of reprocessing methods only indicates that BK Medical has evaluated the transducers for material compatibility, when reprocessed according to the product/system IFU, and not for their efficacy in attaining the appropriate level of reprocessing. Ensure that the reprocessing chamber is an appropriate size for the transducer.

b) Transducer connector is not immersible

c) Follow product manufacturer's instructions and do not exceed transducer-specified limits. See Care and Cleaning Guide for more information.

d) STERIS SYSTEM 1, SYSTEM 1 PLUS, and SYSTEM 1 Express have not been market cleared by the FDA.

e) Transducer has not been licensed by Health Canada

Information in this document is subject to change without notice.

Printed on 09/08/2020